

Provox Life™ Standard Adhesive Technical Data


Provox Life™ Standard Adhesive

Technical Data


Provox Life™ Standard Adhesive





Provox Life™ Sensitive Adhesive

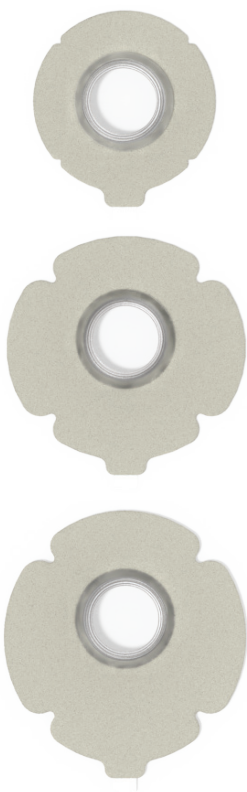

Provox Life™ Night Adhesive

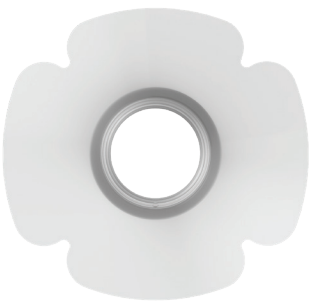

Provox Life™ Stability Adhesive

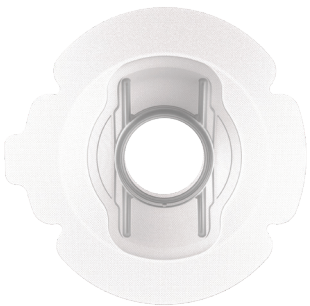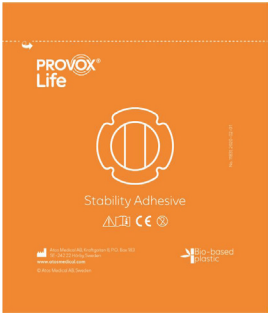

Provox Life™ Standard Adhesive

Technical Data

Carrier:

Polyethylene. Perforated.

Adhesive:

Medical Grade Acrylic

Liner:

Paper with siliconised surface. One piece.

Connector:

SecureFit. Diameter 23mm.

Application:

The pressure sensitive medical grade acrylic material reaches maximum adhesion after about one hour and does not benefit from pre-warming prior to application.

Removal:

Reduced moisture transport rate (MTR) may lead to higher levels of moisture trapping underneath the adhesive which helps to reduce stickiness for adhesive removal. Use Provox Adhesive Remover wipe on top of perforated carrier to saturate material and loosen adhesive prior to removal. Then use adhesive remover wipe from the side to gently remove adhesive – starting from finger lift tab.

Technical data versus legacy device Provox Flexiderm adhesive:

The materials of this adhesive are the same as for the legacy Provox Flexiderm device. However, the coat weight (amount of adhesive (i.e. glue)) is 67% higher which increases stickiness, lowers permeability, and reduces MTR. Lower permeability and MTR reduces blow-through (i.e. air blowing through the adhesive material in spots where it is not connecting with the peristomal skin) during TE speech.

Lab tests demonstrated a >400% reduction in blow-through.

Sizes:

Comes in 3 sizes – Round, Oval, Plus

Provox Life Standard Adhesive Provox Life Stability Adhesive	Material: Medical grade acrylic
Flexibility	High
Permeability	Moderate
Absorption	None
Adhesion	High tack
Risk of skin stripping	High, if not removed correctly